## INSULIN DELIVERY DEVICE

# Quick Guide


For use only with HUMALOG® or HUMULIN® 3 mL Insulin Cartridges (100 units/mL)

www.lilly.ca

This guide is a quick reference and does not replace the full User Manual. Read the full User Manual completely and follow the directions carefully before using your new HumaPen MEMOIR®.

If your HumaPen MEMOIR does not operate as described by this Quick Guide or you have questions, refer to your pen User Manual, contact your healthcare professional or call Lilly at 1-888-545-5972 for assistance.


### ON Button Functions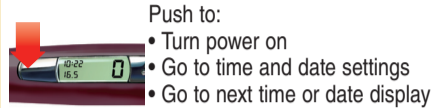

Push to:

- Turn power on
- Go to time and date settings
- Go to next time or date display

### Dose Knob Functions

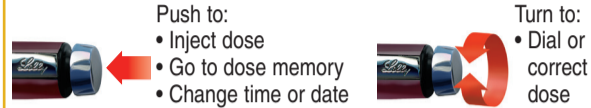Push to:

- Inject dose
- Go to dose memory
- Change time or date

Turn to:


- Dial or correct dose

## SETTING TIME AND DATE

In READY MODE, press and hold ON button for 3 seconds. The hours number will start flashing.

To go to the time or date mode you want, press the ON button repeatedly until the desired mode is flashing.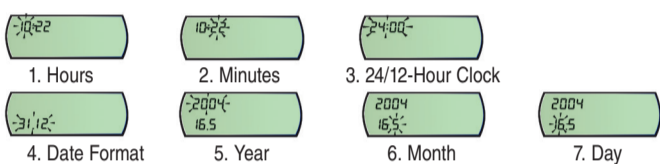

Press the ON button again to exit and return to READY MODE. The changes will be saved.


## LOADING OR CHANGING A CARTRIDGE

### a Unscrew Cartridge Holder

Remove empty cartridge.

### b Insert Insulin Cartridge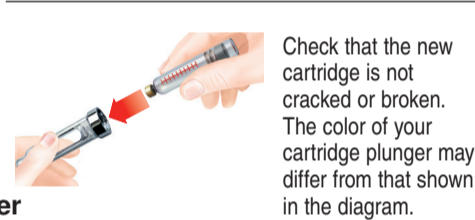

Check that the new cartridge is not cracked or broken. The color of your cartridge plunger may differ from that shown in the diagram.

### c Screw Cartridge Holder and Pen Body Together


Use cartridge plunger to push injection screw back.

Push straight together and screw until secure.

## PREPARING AND PRIMING YOUR PEN

### d Inspect Insulin

For all insulin types:

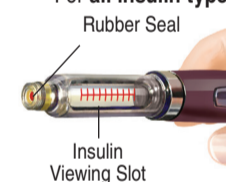Look through the insulin viewing slot and check that the cartridge label is for the correct type of insulin.

For cloudy insulin only:

To evenly mix insulin, gently roll pen 10 times,


and invert pen 10 times.


### e Attach Needle

Pull off paper tab.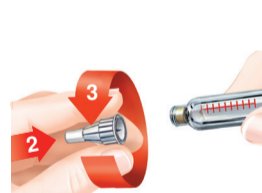

Screw needle straight on until secure.

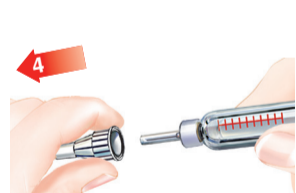Pull off outer needle cap and keep.

### f Power On and Display Check

Press ON button to turn pen on. Pen will automatically turn off after 30 seconds of inactivity.

When the pen is turning on, you should see the following displays:

2 SECONDS


1 SECOND

READY MODE  
Time, date and 0 in display.

### g Prime Every Time Before Injecting

DIAL  
2 UNITS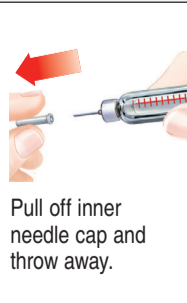


Pull off inner needle cap and throw away.

READY MODE


After priming, display will go to READY MODE. You may need to prime several times until a stream of insulin flows from needle.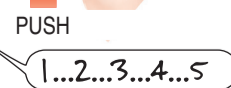

PUSH  
1...2...3...4...5

## SETTING AND INJECTING YOUR INSULIN DOSE

### h Dial Your Dose

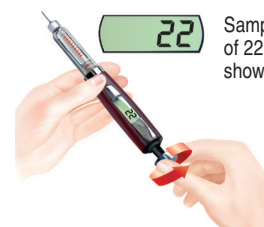Sample dose of 22 units shown.

If you dial too many units, you can correct the dose by dialing backwards.

**Do NOT** use the pen if the numbers in the display are incomplete or do not dial in order (1, 2, 3...)


**Do NOT** set your dose by counting dial clicks if you do not see dose numbers in the display. You may receive the wrong dose.

### i Inject and Confirm Your Dose

Insert needle.

Place thumb directly on the dose knob.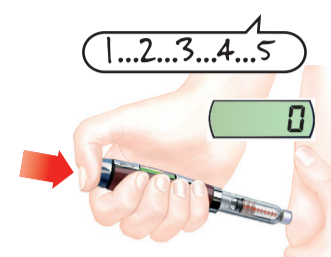

Slowly push straight on dose knob as shown until dose knob stops. When the dose knob stops, continue to hold it in and count to 5 slowly.

READY MODE

Remove needle from skin. After injection, pen should be in READY MODE.


**Note:** If the display does not show the READY MODE, the number in the display is the amount you did NOT receive. Do NOT continue to push dose knob. To get the rest of your dose, go to *Finishing an Incomplete Dose*. Remove needle and push cap on pen.

If you do not think you received the full amount you dialed for your injection, do not repeat your injection. Call your healthcare professional or Lilly at 1-888-545-5972.

## FINISHING AN INCOMPLETE DOSE

HumaPen MEMOIR allows you to dial your full dose amount even if there is not enough insulin in the cartridge.

When the injection is complete and after you have released the dose knob, you will see a number in the display. Remember this number. This is the amount of insulin you did NOT receive.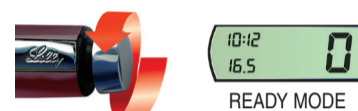

Dial dose knob down to 0. This will save the first injection in dose memory and return pen to READY MODE.


To get the rest of your dose:

- Change insulin cartridge
- Inspect insulin and prime
- Dial remaining dose
- Inject and confirm dose

**Note:** Your dose memory will record this as a separate dose.

## VIEWING DOSE MEMORY


HumaPen MEMOIR has a dose memory that allows you to see the time, date and amount of the last 16 doses that were delivered including the priming doses. To see stored doses:

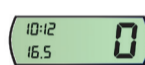READY MODE  
Make sure pen is in READY MODE.

Press and release dose knob to go into dose memory. You will see your most recent dose first.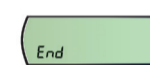

Press dose knob repeatedly to see all stored doses until 'End' is displayed.

After 'End' is displayed, press dose knob again to return to READY MODE.

**Note:** If dose knob is not pressed, pen will return to READY MODE after 15 seconds. Your HumaPen MEMOIR is tested at the factory and may contain several stored doses when you first receive it.

## DISPLAY MESSAGES AND ERRORS

### DISPLAY

### WHAT IT MEANS


### WHAT TO DO

#### ERRORS

Dose knob dialed past 60 units


- Dial dose knob backwards until 60 shows in display. Do NOT dial dose knob past 60.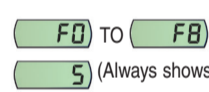

Permanent error has happened

- Do NOT use pen. Contact your healthcare professional or Lilly at 1-888-545-5972.

#### BATTERY

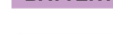Battery is low with less than 3 weeks of life left

- Contact your healthcare professional or Lilly at 1-888-545-5972 for a new HumaPen MEMOIR.
- Pen can be used until dead battery shows in display.

Dead battery

- Do NOT use pen. The batteries are not replaceable; contact your healthcare professional or Lilly at 1-888-545-5972 for a new HumaPen MEMOIR.

#### PEN OPERATION


Display check. Confirms all display parts are working when pen is turned on.


- No action needed.
- If any part is missing during 2-second display, or if any part is lit during 1-second blank display, do NOT use pen. Contact your healthcare professional or Lilly at 1-888-545-5972.

1 SECOND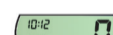


READY MODE

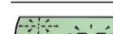Reset mode  
A temporary error may have occurred.

- Do NOT inject while in Reset Mode as the dose setting will not be shown on the display and the dose delivered will not be stored in the memory.
- If the pen has been stored in a cold environment (e.g. refrigerator), allow the pen to warm to room temperature.
- Dial dose knob down all the way and then press and hold dose knob down for one second. Pen will reset and go to READY MODE. If the pen will not reset, do NOT use the pen. Contact your healthcare professional or Lilly at 1-888-545-5972.

#### DOSE MEMORY

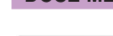
Dose memory display (Time, date and number other than 0 in display)

- No action needed.

At the end of stored doses

- Press dose knob to go to READY MODE.

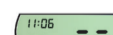Dose was not stored (invalid memory)

- You may have injected your last dose too quickly. Contact your healthcare professional or Lilly at 1-888-545-5972 if this error persists.

**Prime every time before injecting.** Priming fills any empty spaces inside the needle or cartridge with insulin and checks that insulin will flow out of your pen when you inject. You may need to prime several times to get a stream of insulin coming out of the needle. If you do NOT prime, you may get too much or too little insulin.

Keep the pen away from dust, extreme hot or cold (freezing) temperatures and direct sunlight.

The pen has been designed to operate from 5°C to 40°C (41°F to 104°F). For maximum pen life, store at room temperature.

Do NOT store the pen in the refrigerator. Condensation may form inside the pen, which can damage the electronics.

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